

# Replacing the Strap on Your Arm Rest Shelf

If you have not yet attached the Lowered Arm Rest Shelf to your table, please refer to the "Installation of Your Lowered Arm Rest Shelf" instruction sheet.

Slide the QuickLock™ Face Rest into the table. Lock the face rest into a

slot of the arm rest strap into the strap slot from the front (Fig. 2) of the table until it all the way through until the buckle fastened to the other end of the slot. Be sure that the writing printed on the buckle is facing the front padding. See Fig. 1 & 2

Turn the rest shelf over so that the back is facing you. Slide the slip buckle into the slot opening approximately 8 inches from the slot. Thread the webbing through the slip buckle as shown in Fig. 3 & 4

Slide the strap back through the strap slot. You will see that the slip buckle is now in place and keeps it from pulling through the slot. Fig. 7

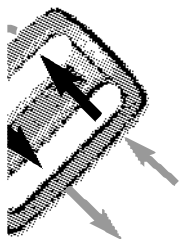


Fig. 4

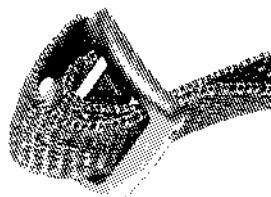


Fig. 6

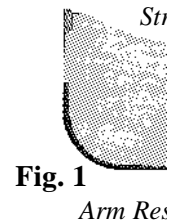
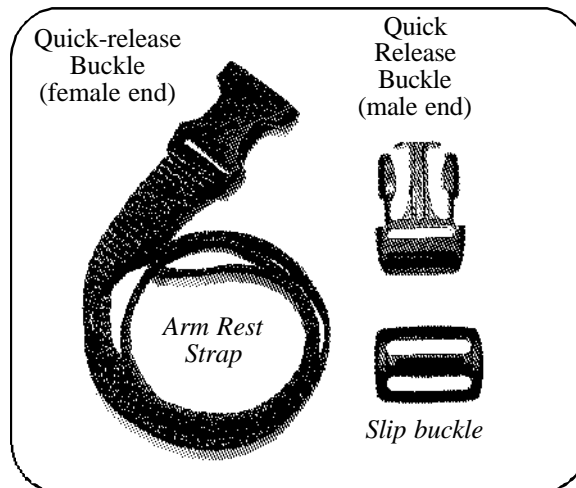
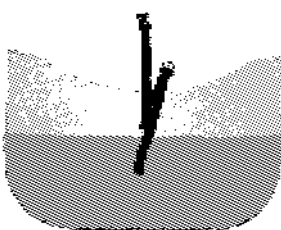


Fig. 1  
Arm Rest

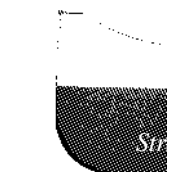


Fig. 2  
Arm Rest

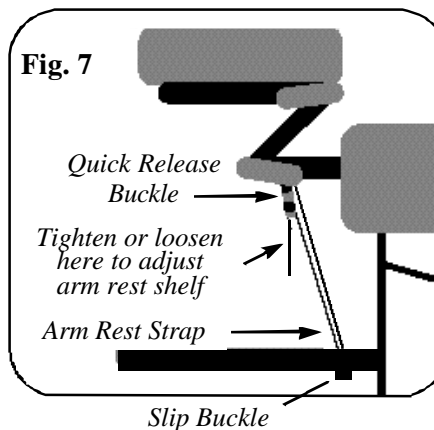


Fig. 7

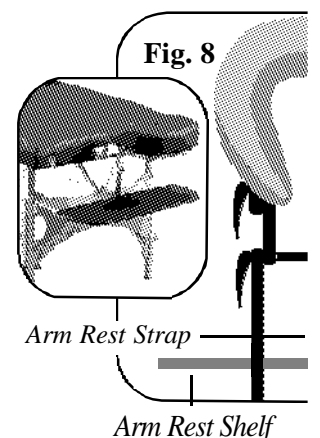


Fig. 8

6. Next, thread the strap through the male end of the quick release buckle in Fig. 5. When this is done the arm rest should look like Fig. 6.

7. Snap the two ends of the quick release buckle together around the QuickLock™ face rest. See Fig. 8

8. To adjust your Arm Rest Shelf, pull on the strap to raise the Arm Rest Shelf. To loosen the strap to lower it. See Fig. 7. When completed your arm rest shelf is attached to your table as seen in Fig. 8 inset.

**CAUTION:** Excessive force, such as clients pushing on the Arm Rest to lift the table may damage the table as well as the arm rest. This type of damage is not covered by your warranty.

For further assistance please call Customer Service at: 1.800.558.8850 or 717.235.6807  
www.oakworks.net




equipment

U.S. Patent #5,177,823  
part no. 4228

7/00

# Attaching the Arm Rest Shelf to the QuickLock Face Rest

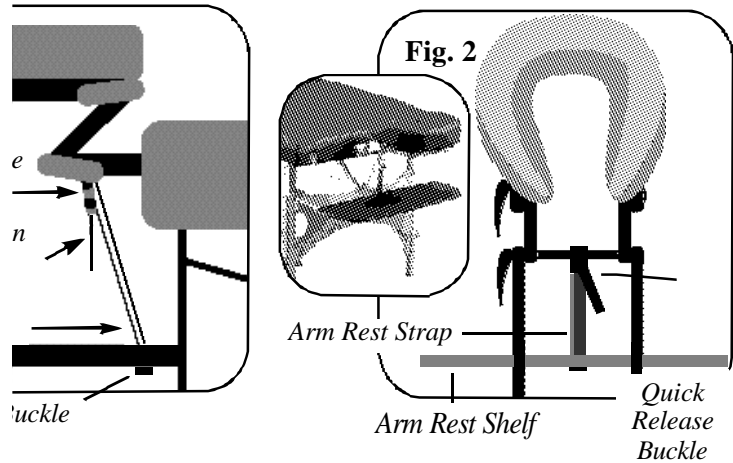


ends of the *quick release buckle* together around the cross bar of the QuickLock™ face rest. See *Fig. 2*

To raise or lower the Arm Rest Shelf, pull on the strap to raise the Arm Rest Shelf or pull on the strap to lower it. See *Fig. 1*. When completed your arm rest will be attached to your table as seen in *Fig. 2 inset*.

Excessive force, such as clients pushing on the **Arm Rest** to lift themselves off the table may damage the table as well as the arm rest. This type of damage is not covered by your warranty.

# Attaching the Arm Rest to the QuickLock Face Rest

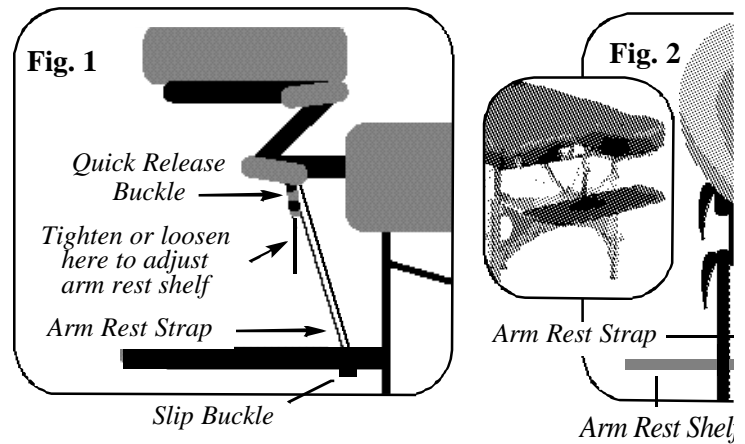

1. Snap the two ends of the *quick release buckle* together around the cross bar of the QuickLock™ face rest. See *Fig. 2*
2. To adjust your Arm Rest Shelf, pull on the strap to raise the Arm Rest Shelf or pull on the strap to lower it. See *Fig. 1*. When completed your arm rest will be attached to your table as seen in *Fig. 2 inset*.

**CAUTION:** Excessive force, such as clients pushing on the **Arm Rest** to lift themselves off the table may damage the table as well as the arm rest. This type of damage is not covered by your warranty.